

# Analog Denture Technique

## No Reference Denture

### Initial Appointment

1. Make final impressions
2. Complete the RX order form and ship the impression to AvaDent for base plate with wax bite rim.
3. If a scanner is available at the clinic, scan impressions, complete the Rx order form, and send scan data to AvaDent to fabricate wax rims.

### Wax Rim Appointment

1. Verify fit. Adjust and/or take wash impression as needed
2. Seat wax rims to verify the vertical dimension and occlusal.
3. Verify/Modify occlusal plane
4. Mark midline, smile line, and canines.
5. Make bite registration
6. Scan each wax rim 360 degrees
7. Scan wax rims together in bite
8. Upload the files to AvaDent for design and fabrication of a try-in.


### Try-In appointment

1. Verify fit; adjust and/or take wash impression if needed.
2. Seat try-in and confirm midline, incisal edge, VDO, lip support, esthetics, and bite.
3. If all records are correct, submit to AvaDent for final denture.
4. If changes are needed, repeat Wax Rim steps and upload files for redesign.

### Final Fit

1. Seat final denture fabricated by lab.
2. Perform fit adjustments and patient education.
3. Schedule follow-up for sore spot relief if needed.

#### ANALOG IMPRESSION

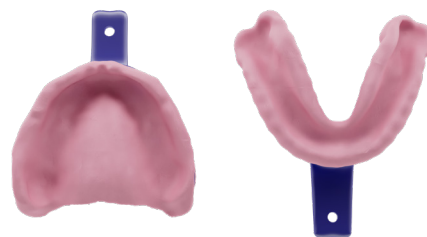

#### BASE PLATE W/ BITE RIM



#### AVADENT TRY-IN



Full Color Printed Try-In

#### FINAL DENTURE | XCL-2



Fully Milled Denture - Multi-Layered Teeth



**Analog Denture Technique**  
**No Reference Denture**  
 Scan QR To Watch Video