


Analog Overdenture Technique

Removable Zest LOCATOR® System

Initial Appointment

1. Take abutment-level impression over LOCATORs
2. Capture soft tissue around abutments precisely
3. Remove all housings before scanning or shipping
4. If possible digitally scan impressions and upload with RX
5. If no scanner, ship impressions to AvaDent

ANALOG IMPRESSION

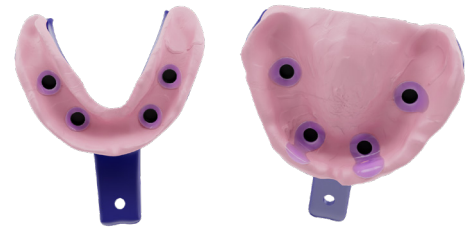

Wax Rim Appointment

1. Verify fit. Adjust and/or take wash impression as needed
2. Seat wax rims with LOCATOR pockets to verify vertical dimension and occlusion
3. Record midline, smile line, and occlusal plane
4. Capture bite registration

BASE PLATE W/ BITE RIM



Try-In Appointment

1. Try-in full-color prototype denture with locator pockets with LOCATOR housings in place
2. Verify fit. Adjust and/or take wash impression as needed
3. Verify esthetics, phonetics, and occlusion
4. Make adjustments as necessary and confirm with patient

FULL-COLOR TRY-IN



Final Fit

1. Deliver AvaMax Overdenture with LOCATOR recesses or standard AvaDent overdenture
2. Use chairside pickup technique to engage housings
3. Confirm passive fit, retention, and provide patient care instructions

FULLY MILLED OVERDENTURE



With or Without Titanium Frame



Analog Overdenture Technique
Removable Zest LOCATOR System
 Scan QR To Watch Video